


FetaLink is an application used by clinicians to monitor maternal and fetal data during the different phases of labor. FetaLink allows for data to be displayed in a meaningful and useful format to clinicians in the ambulatory and inpatient settings.

Documentation Integration

Ambulatory practices will document the Fetal Monitoring episode after the Non-Stress Test order has been placed from the WH Quick Order MPage by entering a **Reason for Monitoring** and any necessary nursing annotations in FetaLink.


Documentation in FetaLink

- Documenting **Reason for Monitoring**:
 - Associate the patient to the monitor.
 - From the Census view, click the column **Reason for Monitoring**.
 - Associate the patient to the correct encounter.
 - Click **Reason for Monitoring**.
 - Click the reason for monitoring from the provided list.
 - Click **OK**.

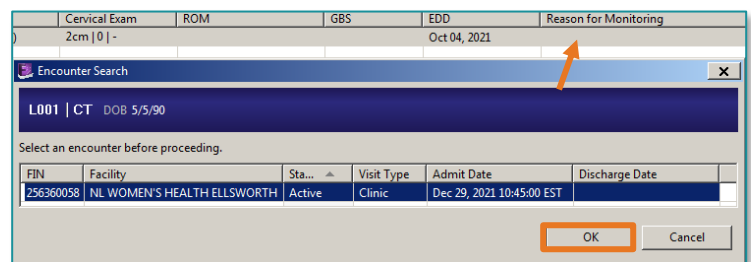

NOTE: During the monitoring episode, address any alerts and create an annotation.

- Creating an Annotation for an alert:
 - Click the paper clip icon.
 - Select options from the quick picks.
 - If the options aren't appropriate, free text in the annotation box.

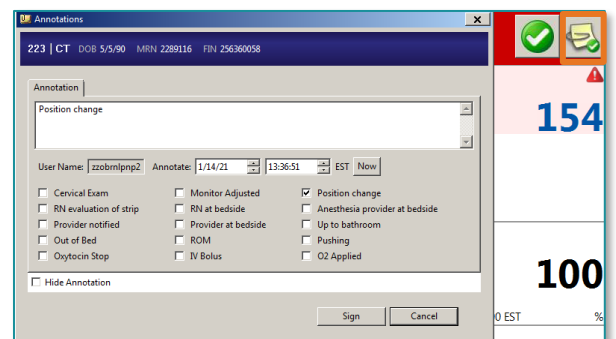



➤ Documentation in PowerChart from FetaLink

- Annotations (flow automatically) to the FHR Annotations Field found in the Antenatal Testing Band of Interactive View and I & O.

		01/14/2021	
		13:44 EST	13:36 EST
Fetal Monitoring Annotations			
Fetal Monitoring Annotations			Position change [2]



➤ Documentation from PowerChart to FetaLink

- The provider will document their exam/interpretation in the Antenatal Testing Band in Interactive View and I&O.
- This information will display in FetaLink as an Annotation.

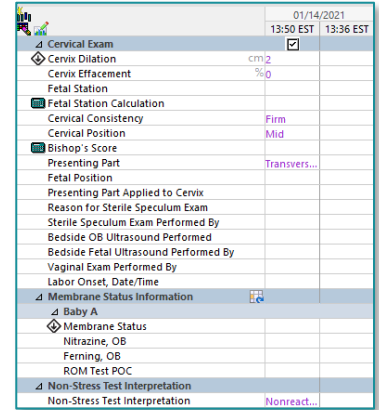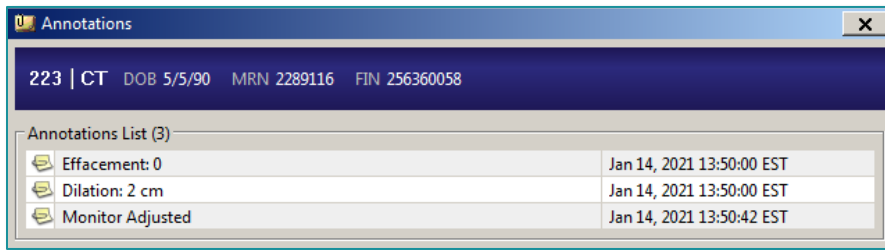




➤ Finalizing the Fetal Monitoring Episode

- When the fetal monitoring episode is complete, the patient will be taken off the monitor and the episode will be finalized.
- Select **Finalize**.
- Review/Update the **Reason for Monitoring** and select **Yes**.

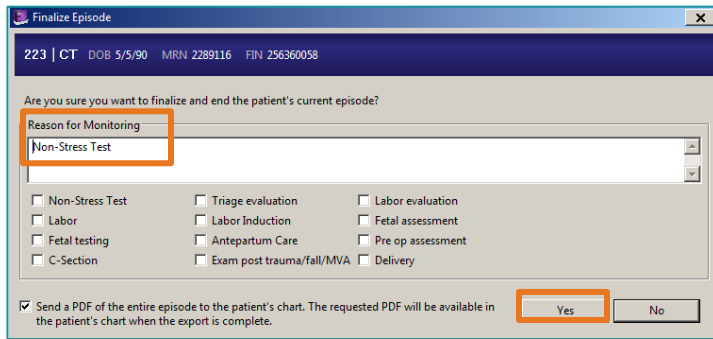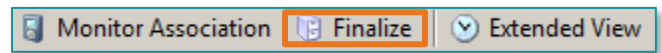


➤ Single Patient Task List

- After the fetal monitoring episode has been finalized, review the patient's **Single Patient Task List**.
- From the drop down, select **Done**.
- Complete the form, as appropriate, and click the green checkmark to sign the form.

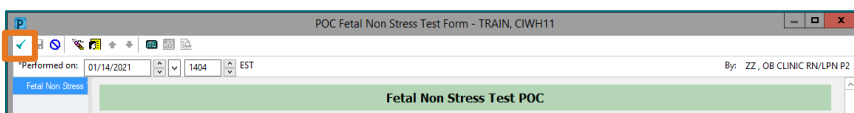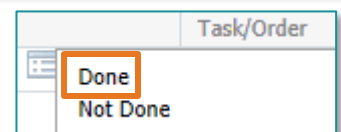
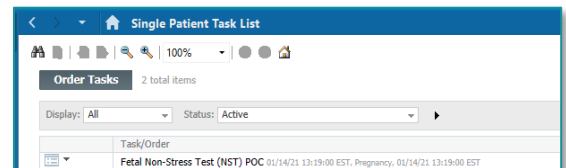


➤ **Antenatal Testing Note**

- From the Prenatal MPage, select the **OB Antenatal Testing Note**.
- If being initiated by nursing, nursing will review the information and **Save the Note**.

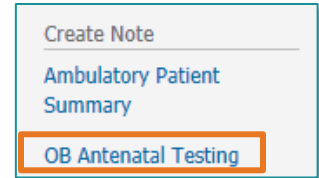


- Navigate to Notes (on the Table of Contents), locate the note initiated by nursing and click the person icon.
- Forward the note to the provider for signature by selecting Sign in the Additional Forward Action drop down.
- The provider will review the note in their Message Center prior to signing.

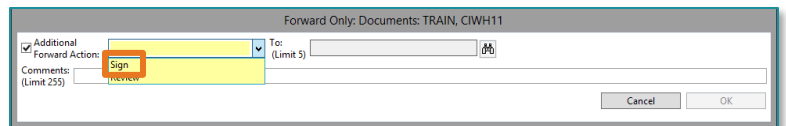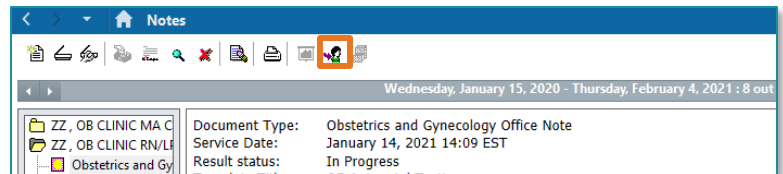

➤ **Viewing the Fetal Monitoring Episode**

- From the Prenatal MPage, select **Fetal Monitoring**.
- Click the episode that you want to view and select **Launch**.
 - This will take you to FetaLink to view the fetal monitoring strip.

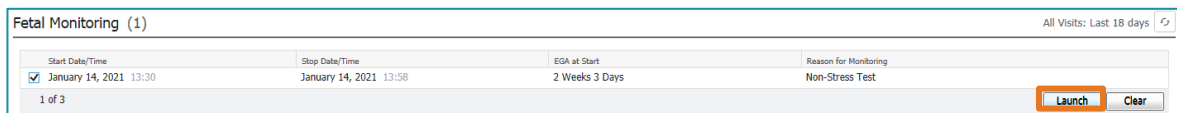

- To view a PDF version of the fetal monitoring episode, select **Media Gallery**.
 - Locate the FetaLink PDF and click the episode(s) you want to view.

