

Bridge Breast Milk Management will allow end users to record the amount of milk administered. With the Combine Bottles process, more than one bottle can be scanned when preparing for a feeding. There are no limits on the number of times a bottle can be scanned or on the number of bottles that can be scanned at the same time.

Combining Bottles

STEP 1: Identify the patient by scanning the patient’s ID from their wristband or by scanning the patient’s storage label.

STEP 2: Select Combine Bottles.



STEP 3: Scan the bottle number from the bottle label on the container. The system verifies the bottle number can be combined for the patient.

STEP 4: If required, enter a thaw date and time.

NOTE: This step is required if Expiration Date Checking is enabled and the bottle is in a Frozen state with no thaw date and time.

- If the Volume box is available, you can enter the bottle volume. The maximum value that can be entered is 99999. Decimal values can have one digit after the decimal. The system stores the bottle and the volume is entered in milliliters (ml).
- Continue scanning or entering bottles as need. At least two bottles are required for this process.

						<input checked="" type="radio"/> New Bottle
						Total Combined: 2
Dispose	Bottle #	Expires	State	Fortifiers	Combine Into Bottle	
<input type="checkbox"/>	1050	6/12/2021 11:20 EDT	FRESH		<input type="radio"/>	
<input type="checkbox"/>	1051	6/12/2021 11:20 EDT	FRESH		<input type="radio"/>	



➤ Determine if any bottles need to be disposed. If all bottles are selected for disposing, the system generates a new bottle number and prompts you to select a printer when you select **Prepare** or **Exit**.

➤ Select the **New Bottle** button if you are creating a new bottle. If you are combining milk into an existing bottle, select the **Combine Into Bottle** button. The option to dispose the bottle being combined into becomes unavailable.


						<input type="radio"/> New Bottle
						Total Combined: 3
Dispose	Bottle #	Expires	State	Fortifiers	Combine Into Bottle	
<input checked="" type="checkbox"/>	1050	6/12/2021 11:20 EDT	FRESH		<input type="radio"/>	
<input type="checkbox"/>	1051	6/12/2021 11:20 EDT	FRESH		<input checked="" type="radio"/>	
<input type="checkbox"/>	1052	6/12/2021 11:20 EDT	FRESH		<input type="radio"/>	

NOTE: The Combine Into Bottle button is displayed for the scanned bottle if the volume entered is the same as the full volume present in the bottle. If you enter a volume other than the present full volume of the bottle, the Combine Into Bottle button is not available.

- Enter any comments. Comments are displayed on the bottle label (the new bottle being created, or the bottle being combined into) and on the **Browse Bottle History** page.
- If you select **Prepare**, the Prepare Bottles page is displayed. A label is not generated.

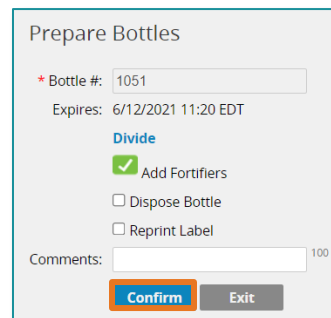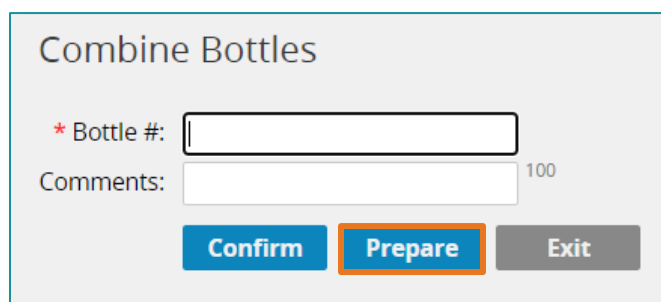

- If you select **Confirm**, either the printer selection page or the main menu page is displayed.
 - The printer selection page is displayed in either of the following scenarios:
 - You are creating a new bottle and selected the **New Bottle** check box.
 - You selected the **Combine Into Bottle** check box and the contents of the other bottles are different from the selected bottle (for example, a different fortifier or expiration date).
 - The main menu page is displayed in either of the following scenarios:
 - No new bottle is being created.
 - You selected the **Combine Into Bottle** check box and the contents of the other bottles is the same as the selected bottle.

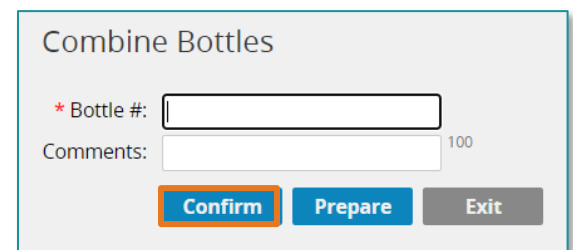

NOTE: Selecting **Exit** does not combine any bottles or save any bottles scanned. The system displays the main menu page.