


# ECG Management™ - Welch Allyn ECG Cart Workflow

## User Training

---

**\*\*\*\*\*Order driven workflow\*\*\*\*\* CV ECG or CV ECG Pediatric MUST be ordered first. Exception - if patient is emergent and not registered, follow current workflow.**

**Step 1: Go to the ECG Laptop. Start a Resting ECG.**

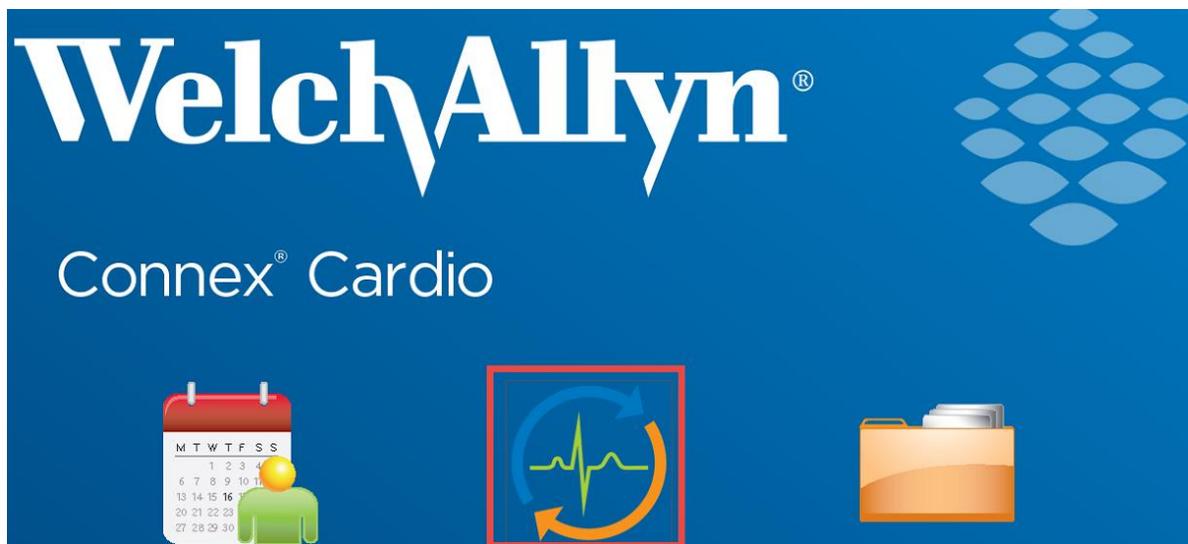

**Step 2: Select the MWL then Search tab.**

Press Search


Patient ID	Last Name	First Name	Date of Birth
191136	Testing	Abigail	
2191134	Testing	ABIGAIL	
2289290	TESTING	ABIGAIL	
0624544	Testing	Alex	

Orders will be returned for 5 days prior and 1 day forward, just like they appear on the ECG Management worklist. Completed exams will not be returned on the worklist. **Be sure to validate the date/time** of the order you select is correct.

Validate patient information. It should be the same as what appeared on the ECG Management worklist.

Patient ID	Last Name	First Name	Date of Birth
191136	Testing	Abigail	
2191134	Testing	ABIGAIL	
2289290	TESTING	ABIGAIL	
0624544	Testing	Alex	

**Step 3: Select the patient from the list and Start Exam.**

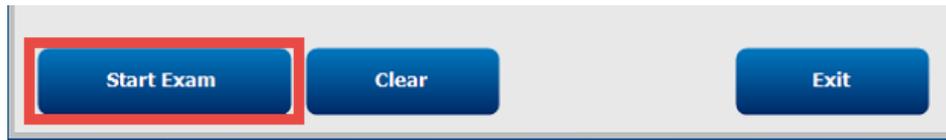

**Step 4: Connect the leads to the patient and press the ECG button on the cart to acquire the ECG.**



**Step 5: Once the ECG has been acquired push the Save button to transmit the ECG to Cerner.**



**\*\*\*\*Please ensure the quality of the ECG Tracing is free of artifact and wander prior to transferring the study to Cerner. \*\*\*\***