


FetaLink is a maternal and fetal monitoring system that facilitates the flow of data from medical devices in a manner that supports display needs of clinicians in the acute and ambulatory setting. When a patient is pregnant with triplets or quadruplets, a second fetal monitor will need to be added.



Adding a Second Fetal Monitor

STEP 1: From the patient's chart within the FetaLink chart, select **Monitor Association**.

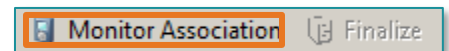


STEP 2: Select the location and **Apply**.


STEP 3: To add the second monitor, scan the second monitor or select the **Monitor Search** link (on the **Monitor Association** box).

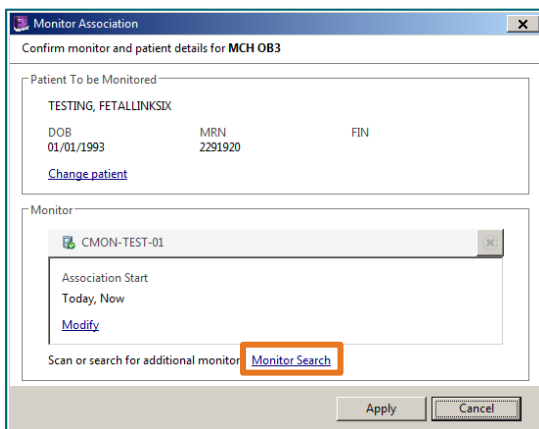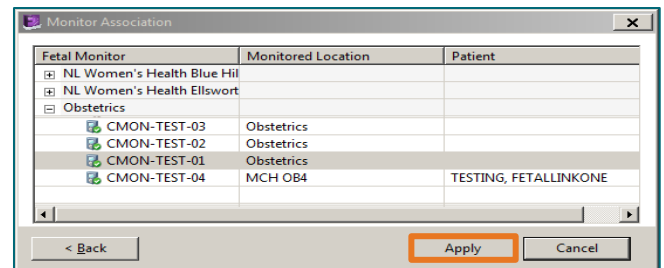

STEP 4: Address the pop-up warning asking if you want to add a second fetal monitor? Click **OK** to proceed with adding the second fetal monitor?

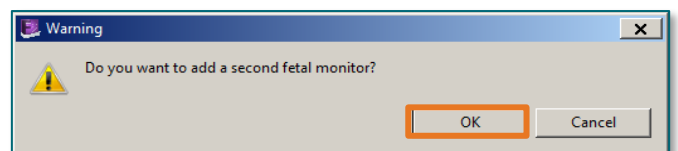

- If adding the second fetal monitor manually, select the correct (2nd) fetal monitor from the list of available monitors and click **Apply** to complete the selection.

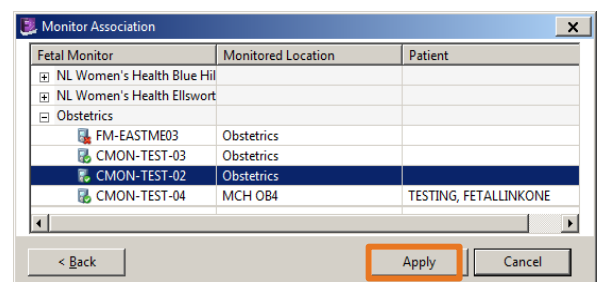

STEP 5: Both fetal monitors will display in the Monitor Association box the patient will be associated with. Verify the monitors are accurate.

STEP 6: Select **Apply** to complete the association.

NOTE: Triplets and Quadruplets require two fetal monitors but will display on one fetal strip.

- Currently FetaLink does not support triplet monitors, so a second monitor will need to be added.
- Both monitors will display in the demographic banner for the patient.

STEP 7: If the encounter wasn't selected when beginning the process, select the encounter for the patient that will be monitored.

