


FetaLink is a maternal and fetal monitoring system that facilitates the flow of data from medical devices in a manner to support display needs of clinicians in the acute and ambulatory setting. The association of the fetal monitor ensures documentation is occurring on the correct patient.

Association of the Patient to the Monitor


STEP 1: Select the location that represents the room where the patient is located.

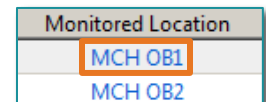


STEP 2: Scan the patient's wristband.

STEP 3: Verify the patient information in the **Monitor Association** window.

- The patient's information will display. If the incorrect patient is displaying, click **Change Patient** and search for the correct patient.

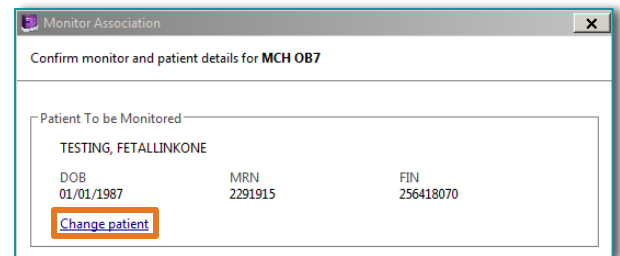


STEP 4: Scan the label on the monitor to associate with the patient.

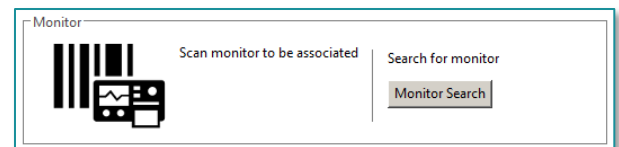

NOTE: If the **Monitor Association** window doesn't display, you are not signed into FetaLink.

STEP 5: Once the wristband and the monitor have been selected, verify the information is correct,

- Information to confirm includes correct location (room number), right patient, right monitor, right time (defaults to today and now).

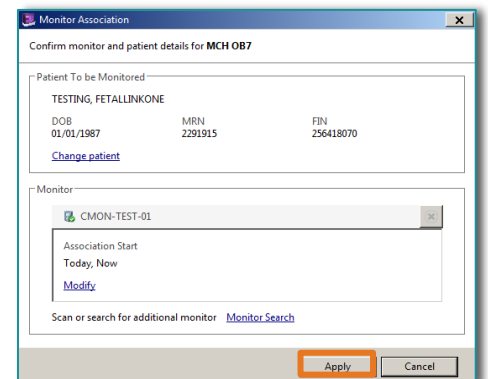


STEP 6: Select **Apply** to complete the association.

Manual Association of the Monitor


STEP 1: From the **Census View**, select the location representing where the patient is located.

STEP 2: From the toolbar in the patient chart, select **Monitor Association**.

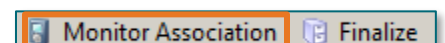

STEP 3: Select **Patient Search** in the **Monitor Association** window.

STEP 4: Enter the patient's FIN (if available) and select **Search**.

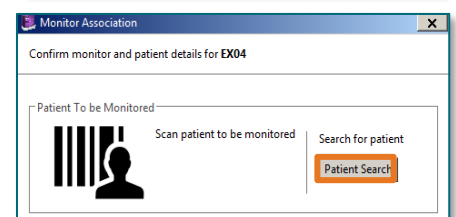

- If the patient's FIN isn't available, a search may be completed by Name, Date of Birth, or Medical Record Number.

STEP 5: After confirming the patient, highlight the patient's name and select **Apply**.

STEP 6: Select **Monitor Search** to launch the monitor search window.

STEP 7: Select the appropriate fetal monitor and select **Apply**.

NOTE: Until an encounter is selected, the patient's FIN and DOB will not display within the demographics bar within the single perspective view, as well as on the Census (FIN).

STEP 8: Select **Apply** to complete the association.

NOTE: When searching by Name, MRN, or DOB, an Encounter Search Box will appear for you to select the correct encounter for this fetal monitoring episode.

The image shows a sequence of three software windows. The top window is a table with patient information:

Name	Date of Birth	Age	MRN	FIN
TESTING, FETALLINKSIX	01/01/1993	28	2291920	256418138

The middle window is titled "Monitor" and contains a barcode icon, the text "Scan monitor to be associated", and a "Search for monitor" button with a "Monitor Search" button below it.

The bottom window is titled "Monitor Association" and contains a table with fetal monitor options:

Fetal Monitor	Monitored Location	Patient
<input type="checkbox"/> NL Women's Health Blue Hill		
<input checked="" type="checkbox"/> CMON-TEST-05	NL Women's Health Blue Hill	
<input type="checkbox"/> NL Women's Health Ellsworth		
<input checked="" type="checkbox"/> CMON-TEST-06	NL Women's Health Ellsworth	
<input checked="" type="checkbox"/> CMON-TEST-07	EX02	TESTING, FETALL
<input type="checkbox"/> Obstetrics		

Buttons at the bottom include "< Back", "Apply", and "Cancel".

The bottom-most window is titled "Encounter Search" and shows a selected encounter:

EX04 | FT DOB 01/01/1991

Select an encounter before proceeding.

FIN	Facility	Visit Type	Admit Date	Discharge Date
256418120	NL WOMENS HEALTH ELLSWORTH	Active Clinic	May 20, 2021 00:00:00 EDT	

Buttons at the bottom include "OK" and "Cancel".