



FetaLink is a maternal and fetal monitoring system that facilitates the flow of data from medical devices in a manner to support display needs of clinicians in the acute and ambulatory setting.

Central Monitor

The Central Monitor at the nurse's station will display FetaLink and is launched from the start menu using the mouse associated with your monitor. At the desk, FetaLink defaults to the Central Monitor View.

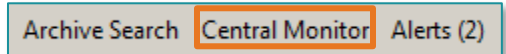


- The Central Monitor view will display any fetal monitor in the unit that is turned on.
- If a monitor is not being used and is turned off, it will not display to save space.
 - Up to 25 monitors can be displayed at a time in the Central Monitor view.

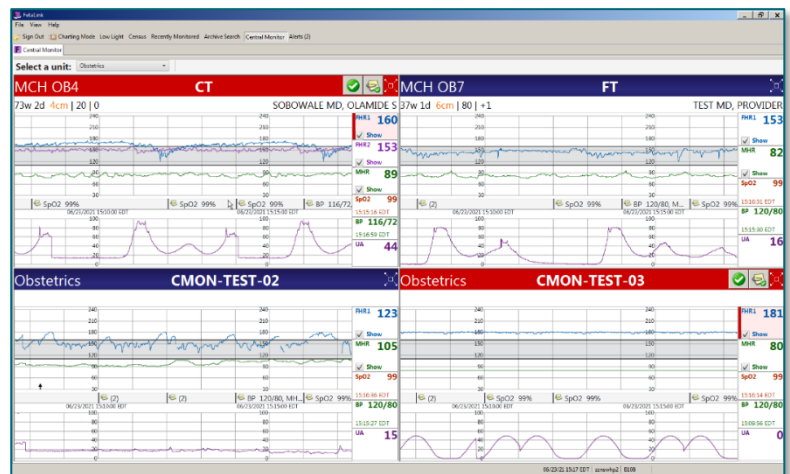

NOTE Different alerts display on the Central Monitor view, unlike the Alerts view which displays only the red alerts (patient alerts).

Alerts View

Alerts View will display all monitors that are in an alert mode for nursing to address.

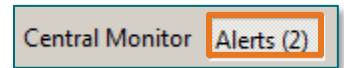

- When Alerts is clicked, the view will change to the strips with alerts present that haven't been annotated or acknowledged.

