

FetaLink is a maternal and fetal monitoring system that facilitates the flow of data from medical devices in a manner that supports display needs of clinicians in the acute and ambulatory settings. There may be times that the association time of the monitor may need to be modified.

## Modifying the Association Time

**STEP 1:** From the toolbar in the patient chart, select the **Monitor Association** button.

- The Monitor Association window will appear.
- The association start date and time that was previously established displays.

**STEP 2:** Within the Monitor Association window, select **Modify**.




**STEP 3:** Update the time field with the correct association time, changing the date as necessary.

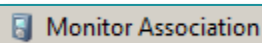
**STEP 4:** Select **Apply** to commit the changes.

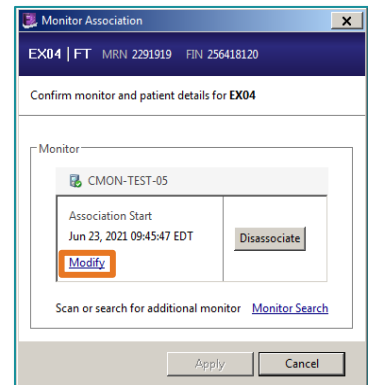
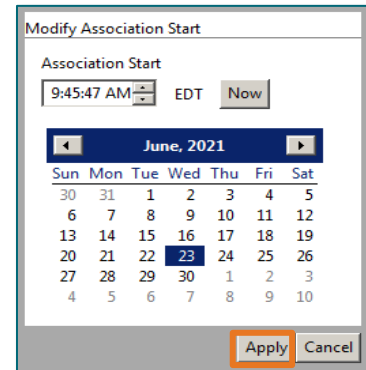
- The updated association time displays.

**NOTE:** The association date/time can be back dated up to 1 week.

**NOTE:** If the user doesn't remember what time the strip started, reviewing the **Annotation Summary** you can find the time the strip was started.

Annotation	Date/Time
 CMON-TEST-01 has been connected.	Aug 25, 2021 16:19:45 EDT
 CMON-TEST-01 has been disconnected.	Aug 25, 2021 16:19:20 EDT
 CMON-TEST-01 has been associated.	Jun 24, 2021 15:56:09 EDT

 Monitor Association

**NOTE:** Selecting **Cancel** will close the window and keep the current association start date/time.