


The Collaborator is a tool within Enterprise Imaging that can be used for communication between the Radiologist or other providers and the Imaging staff.


Locating the Collaborator

STEP 1: Locate the Collaborator in Tools.

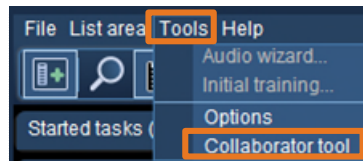

STEP 2: Select Collaborator tool.



- The Collaborator opens.

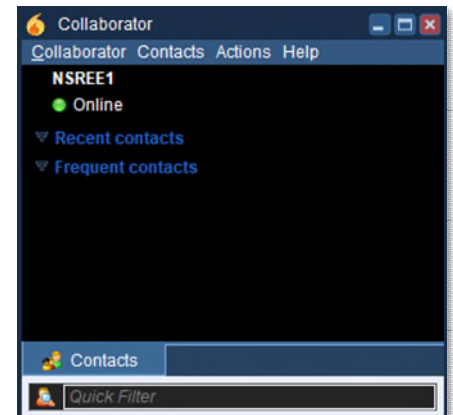

Customizing the Collaborator

STEP 1: Click Collaborator.

- Select **Preferences** to customize the Collaborator.
 - Click the options in the left menu to set preferences for each of the features.
 - Click **Apply**, then **Save** to keep the new preferences.

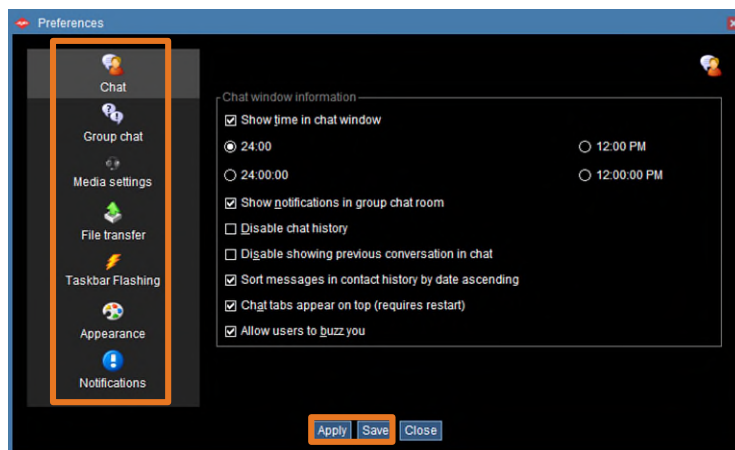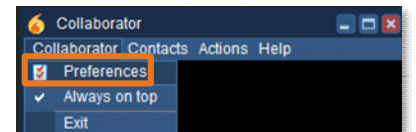


- Unselect **Always on top** IF the Collaborator is not always wanted on the top.
- Click **Exit** to leave the Collaborator.

Adding Contacts


STEP 1: Click Contacts.

STEP 2: Click Add Contacts.

- Enter the Username.
- Select the Username from the dropdown list that displays.

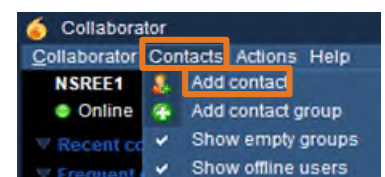


- Select the **Group** the user should be added to using the Group dropdown.
- Click **Add**.
 - The user is added to **Contacted before** and the selected Group.

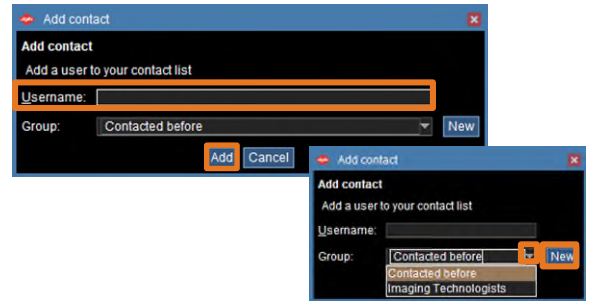

Create a New Group

STEP 1: Click **New** to the right of Group.




- Enter **new group name**.
- Click **OK**.

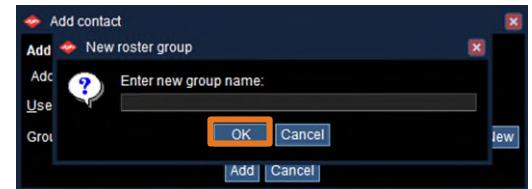

OR

STEP 1: Click **Contacts** and select **Add contact group**.

- Enter **Name of the group**.
- Click **OK**.

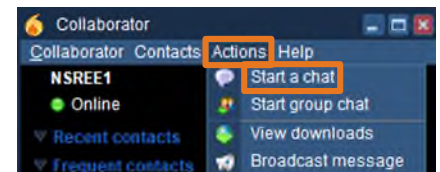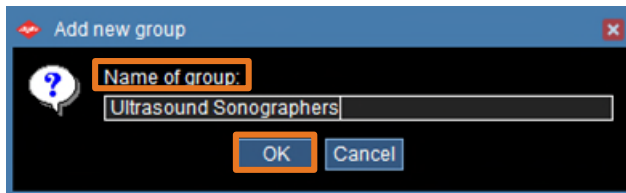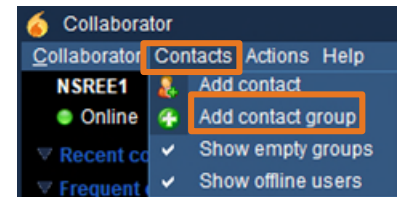


Starting a Chat

STEP 1: Locate the user in the appropriate group.

STEP 2: **Option 1:** Double-click the user's name in a Group.


Option 2: Click **Actions** and select **Start a chat**, enter the username, and click **OK**.

- The **Chat window** opens.
- Type the message.
- Select **Enter** on the keyboard.

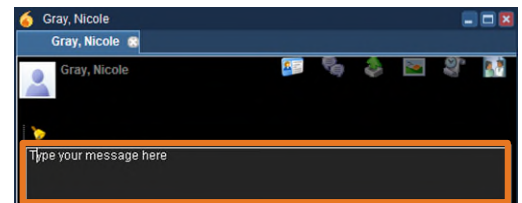





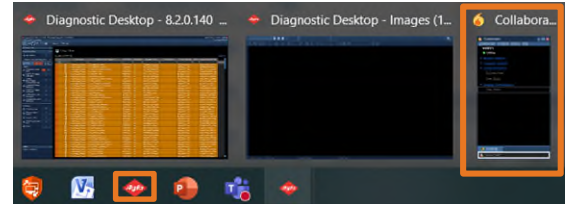

NOTE: Click **Actions** and select **Start group chat** to communicate with more than one person at a time.

- Hover over the **Agfa link** on the Toolbar at the bottom of the screen to reopen the Collaborator.



Additional Chat Window Functionality

- Click  to **view information about the user**.
 - Work email, and phone contact numbers display.
- Click  to **invite to group chat**.
- Click  to **send files to this user**.
- Click  to **capture and send anything on the screen** to the user.
- Click  to **view conversation history**.
- Click  to **get the users attention**.

NOTE: The bell  should NOT be routinely used.