

Viewing PETCT images via XERO Viewer by an image link in the medical record.

**Accessing PETCT Images**

**STEP 1:** From the chart, navigate to the **Imaging** or **Diagnostic** component on a **Workflow MPage**.

**STEP 2:** Select the **imaging study** to be viewed.

**STEP 3:** Click the **View Image** icon. 

**NOTE:** The image status must be **Final Report** to view an image from the EHR.

**STEP 4:** Click the **X** icon to close report.

**STEP 5:** Click the **Cube** dropdown.

**STEP 6:** Click **Extended Tools (PT,CT)**.

**STEP 7:** Click **Hanging Protocol** icon. 

**STEP 8:** Select **PETCT fusion/PETCT fusion AX**.

